




Even better than you think

Self-adhesive absorbent dressing

Mepore is an air permeable, self-adhesive dressing ideal for low to moderately exuding wounds. Its skin-friendly, water-based, solvent-free polyacrylate adhesive provides a gentle and secure fixation.

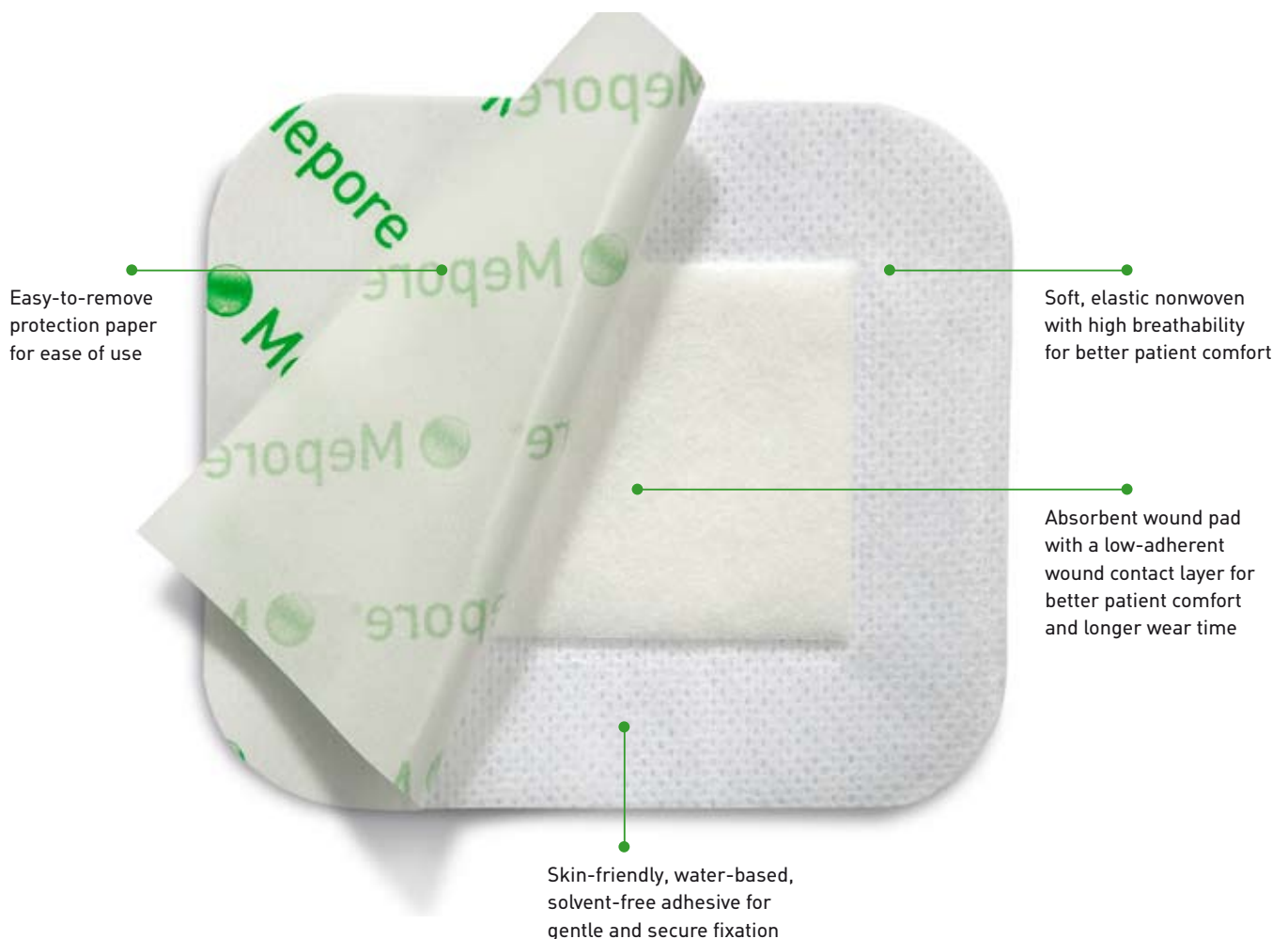

How Mepore works

The porous structure of the nonwoven fabric ensures air and water vapour permeability. The fabric is flexible, conforms well to body contours, and allows body movements when in place. The fabric is non-absorbent and provides a barrier for blood and fluid strike-through. The polyacrylate adhesive has been designed to be gentle to the skin and at the same time provide secure fixation. The wound pad can absorb moderate amounts of wound fluids and the low-adherent

wound contact layer reduces the risk of adherence to the wound.

The easy-to-remove protection paper permits sterile application, as well as simplifies dressing application.

Areas of use

Mepore can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, minor burns, cuts, and abrasions.

How to use Mepore



Open the package and remove the dressing.



Grip both overlapping release papers, exposing adhesive surface allowing initial fixation of the dressing.



Position the dressing on the skin without stretching and gently remove the rest of the release papers. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying.

Frequency of change

The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Benefits of Mepore

- Skin-friendly, water-based adhesive
- Low-adherent wound contact layer minimises risk of adherence to the wound
- Flexible fluid-repellent backing
- Easy-to-remove protection paper allows for convenient and aseptic application
- Air permeable, prevents maceration and gives better comfort
- Rounded corners for secure fixation (all sizes up to 9 x 15 cm)

Precautions

Care should be taken to ensure that Mepore is not applied under tension, to prevent shearing forces causing damage to the skin. This is particularly important when the dressing is applied over joints.



Mepore is packaged sterile in single packs

Art. No	Size cm	Size inch	Wound pad size cm	Wound pad size inch	Pcs/shelf cont.	Pcs/transp. cont.
670800	6 x 7	2.4 x 2.8	3 x 4	1.25 x 2.8	60	480
670870	6 x 7	2.4 x 2.8	3 x 4	1.25 x 2.8	10	140
670900	9 x 10	3.6 x 4	4.5 x 6	1.8 x 2.4	50	400
670970	9 x 10	3.6 x 4	4.5 x 6	1.8 x 2.4	5	100
671000	9 x 15	3.6 x 6	4.5 x 10	1.8 x 4	50	400
671070	9 x 15	3.6 x 6	4.5 x 10	1.8 x 4	5	70
671100	9 x 20	3.6 x 8	4.5 x 15	1.8 x 6	30	180
671170	9 x 20	3.6 x 8	4.5 x 15	1.8 x 6	5	70
671200	9 x 25	3.6 x 10	4.5 x 20	1.8 x 8	30	180
671300	9 x 30	3.6 x 12	4.5 x 25	1.8 x 10	30	270
671400	9 x 35	3.6 x 14	4.5 x 30	1.8 x 12	30	180

Mepore is also available non-sterile in 5 and 2 meter rolls in dispenser boxes

Art. No	Width cm	Width inch	Width of pad cm	Width of pad inch	Boxes/transp. cont.
5 meter (5.5 yard) rolls					
331900	4	1.6	1.5	0.6	32
332000	7	2.8	3	1.2	32
2 meter (2.2 yard) rolls					
331980	4	1.6	1.5	0.6	48
332080	7	2.8	3	1.2	30

Mölnlycke Health Care AB (publ)
Box 13080, SE-402 52 Göteborg, Sweden
Phone + 46 31 722 30 00
www.molnlycke.com

