



mepore®



Even **better** than you think

Improved and developed for increased reliability

When Mepore was introduced in 1979, we actually invented the technique of self-adhesive dressings. To this day, more than 1.5 billion pieces of Mepore have been sold. This shows that Mepore is the one and only “original”, helping nurses all over the world to perform a better wound care.

Mepore has constantly improved and developed during the years. Upgraded adhesive, rounded corners for better fixation and concentrated wound pad with increased absorption capacity are just some recent examples. This is important since our aim is to create dressings that you really can rely on. Both in terms of wound care and cost efficiency.

In year 2000 the showerproof Mepore pro was launched. Three years later we launched an upgraded version with improved performance on critical product properties.

This presentation is just a short reminder of some of Mepore's benefits, and an introduction to some new areas of application.

How to use **Mepore**



Open package and remove the dressing.



Grip both overlapping release papers, exposing adhesive surface allowing initial fixation of the dressing.



Position the dressing on the skin without stretching and gently remove the rest of the release papers. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying!

Frequency of change: The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Areas of use: Mepore can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, minor burns, cuts and abrasions.

Many benefits with Mepore

Water-based solvent-free adhesive. **Gentle to the skin**

Fluid repellent top layer. **Longer wear time**

Highly absorbent pad. **Longer wear time**

Wide border with optimized amount of adhesive. **Secure fixation**



High breathability. **Prevents maceration and gives better comfort**

Flexible fabric. **Better comfort**

Rounded corners. **Secure fixation**


Low-adherent wound contact layer. **Gentle to the skin**

Excellent material strength. **Longer wear time**

Smooth top layer with low friction. **Secure fixation and longer wear time**

More benefits with Mepore pro

Mepore pro offers special benefits. It is showerproof which improves the quality of your patient's life. Furthermore, the viral and bacteria proof backing film gives protection from outer contamination.

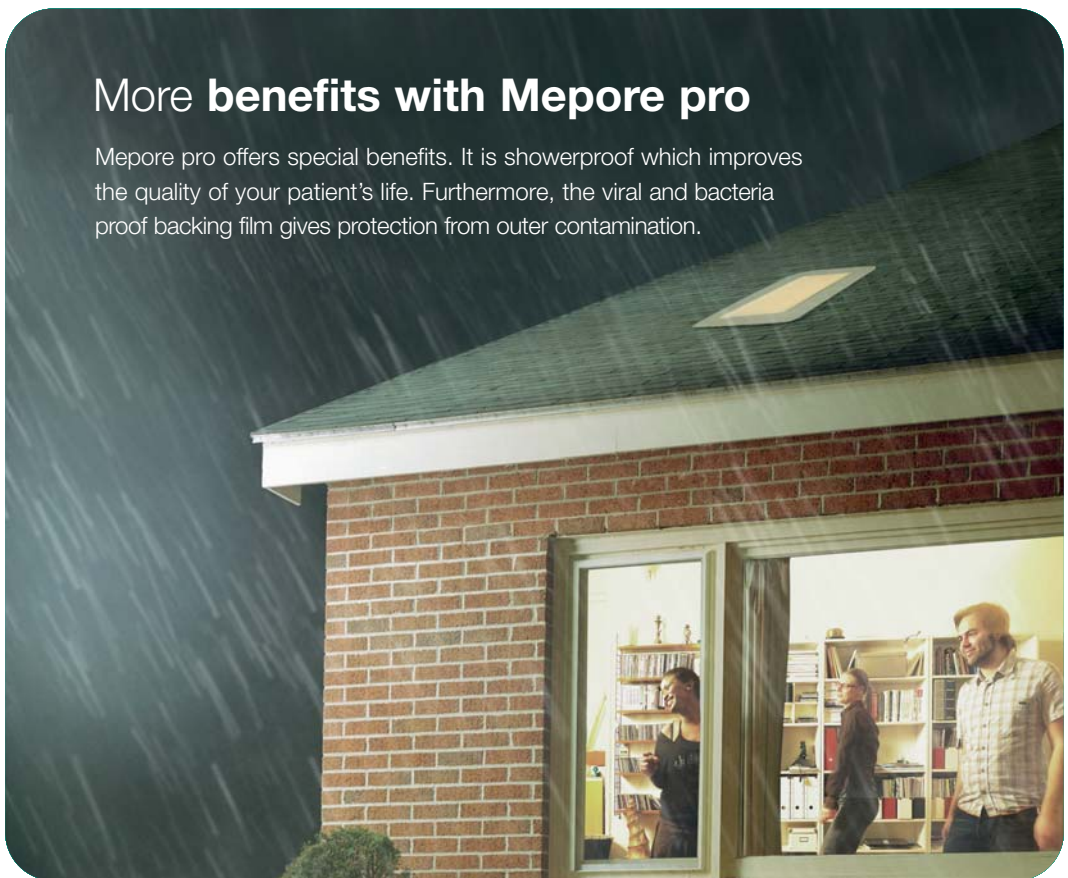



GENTLE CARE™

Mölnlycke Health Care AB (publ), Box 13080, SE-402 52 Göteborg, Sweden,
Phone + 46 31 722 30 00, www.molnlycke.com