Mölnlycke®

Ref no: 139003-06

Created by: Natalie Whiteside

Refer to ARS PD-556208 and Master  
Text IFU PD-582440

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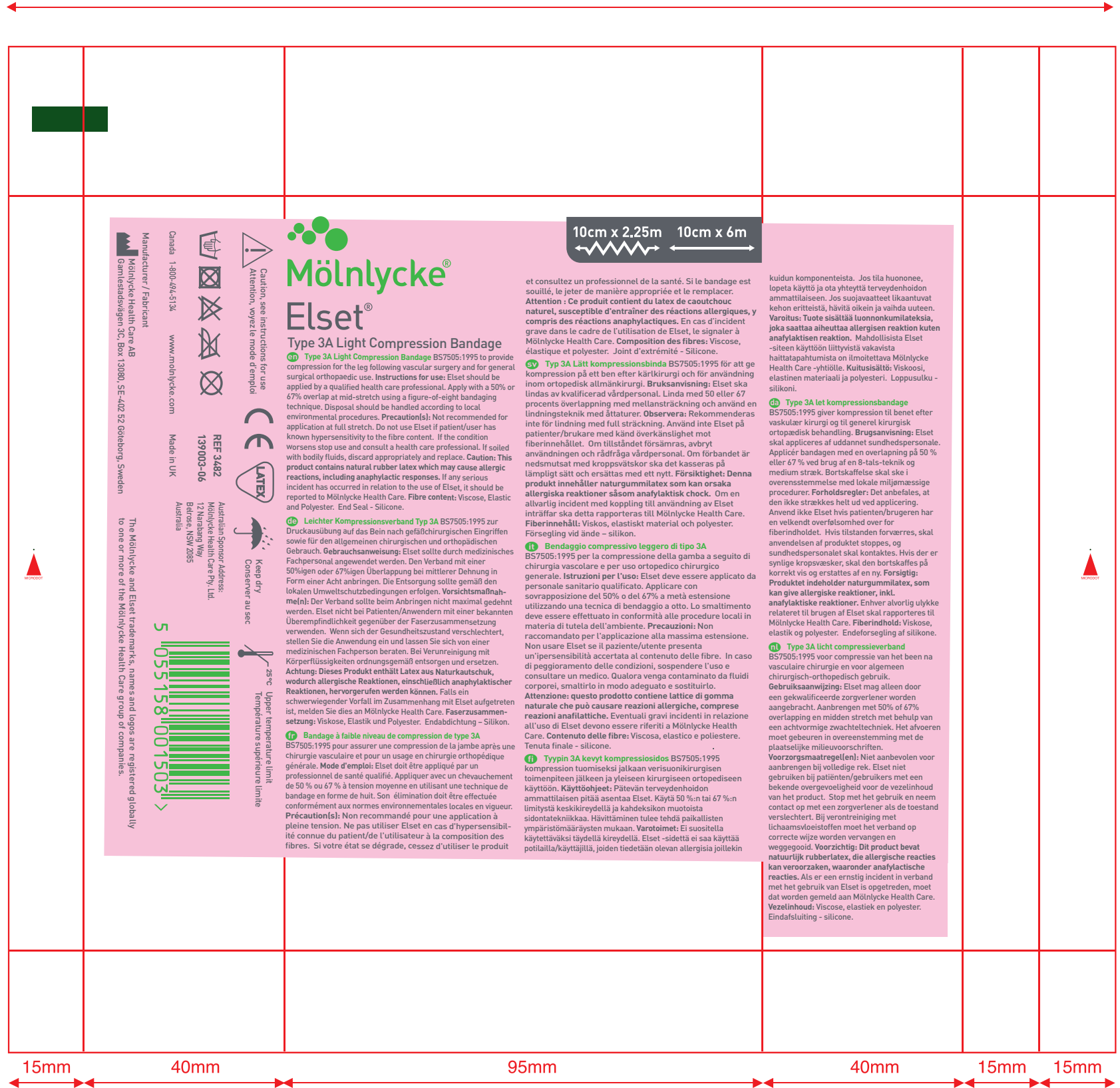
200mm

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Type 3A Light Compression Bandage

**en** Type 3A Light Compression Bandage BS7505:1995 to provide compression for the leg following vascular surgery and for general surgical orthopaedic use. Instructions for use: Elset should be applied by a qualified health care professional. Apply with a 50% or 67% overlap at mid-stretch using a figure-of-eight bandaging technique. Disposal should be handled according to local environmental procedures. Precaution(s): Not recommended for application at full stretch. Do not use Elset if patient/user has known hypersensitivity to the fibre content. If the condition worsens stop use and consult a health care professional. If soiled with bodily fluids, discard appropriately and replace. Caution: This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses. If any serious incident has occurred in relation to the use of Elset, it should be reported to Mölnlycke Health Care. Fibre content: Viscose, Elastic and Polyester. End Seal - Silicone.

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The Mölnlycke and Elset trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies.
Mölnlycke Health Care AB
Mölnlycke Health Care Pty, Ltd.