

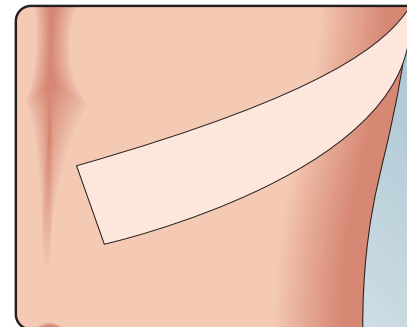
# Mepiform<sup>®</sup>

WITH SAFETAC<sup>®</sup> TECHNOLOGY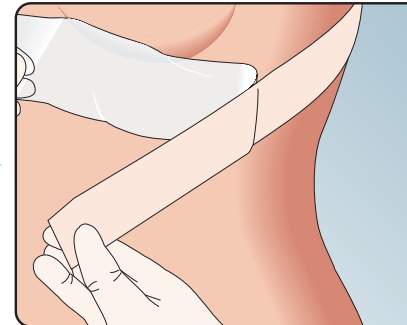

- PREVIENE Y REDUCE LAS CICATRICES
- EVITA E REDUZ AS CICATRIZES
- PREVIENE E RIDUCE LE CICATRICI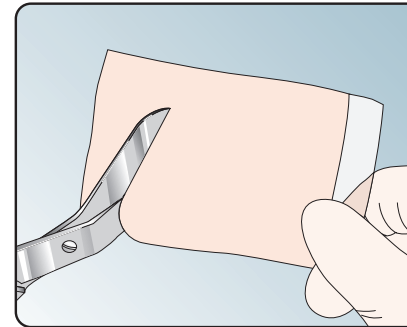

[www.molnlycke.com/symbols](http://www.molnlycke.com/symbols)

UPF Clase 5  
UPF Classe 5  
UPF Classe 5

Impermeable  
Resistente ao duche  
Impermeabile

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## en Mepiform®

### What is Mepiform®?

Mepiform is a self-adherent soft silicone dressing for scar care.

Material content: Polyacrylate, polyurethane, silicone, viscose

### What is Mepiform® for?

Mepiform is designed for the management of both old and new hypertrophic and keloid scars. It can also be used on closed wounds for prevention of hypertrophic or keloid scarring.

### Precautions

- Do not use Mepiform on patients with known hypersensitivity to the ingoing materials/components of the product.
- Should rash occur or the skin become wet and white (maceration), remove the dressing and allow the skin to recover until the symptom has disappeared, then continue treatment gradually increasing therapy time. If the symptom persists, discontinue use and consult your health care professional nurse or doctor.
- Do not use on open wounds.

### Instructions for use

#### Application

- Open the packaging and remove the dressing.
- If needed, cut the dressing to an appropriate size. Mepiform shall cover the scar and surrounding skin by at least 1 cm.
- When applying Mepiform, make sure the area is dry. When used together with ointment or cream, ensure the dressing covers an area beyond the cream.
- Remove the release film and apply the sticky side of Mepiform to the scar without stretching the dressing.

#### Dressing change and removal

- Mepiform should optimally be worn for 24 hours a day. Remove the dressing once per day for inspection and washing of the skin. The dressing can then be reapplied.
- Mepiform should under normal conditions be changed every 7 days or when the adherent properties of the dressing are no longer sufficient.
- Mepiform is showerproof.

Mepiform has an ultraviolet protection factor (UPF) of class 5 in order to protect the scar.

Disposal should be handled according to local environmental procedures.

### Other information

If any serious incident has occurred in relation to the use of Mepiform it should be reported to M lnlycke Health Care and to your local competent authority.

Mepiform® and Safetac® are registered trademarks of M lnlycke Health Care AB.

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Mepiform® e Safetac® sono marchi registrati di M lnlycke Health Care AB.