



Mölnlycke[®] Normasol[®]

STERILE TOPICAL IRRIGATION SOLUTION

Product description

Normasol[®] is a single use sterile isotonic topical irrigation solution contained within an easy to open sachet.

Normasol[®] consists of:

- Sterile 0.9% w/v Sodium Chloride (Ph. Eur.) Solution

Content:

Sodium chloride Ph. Eur., purified water Ph. Eur.

Indications for use

Designed for use in topical irrigation, wound cleansing, irrigation of mucous membranes and rinsing of eyes. Only for external use.

Contraindication(s)

• NOT FOR INJECTION

Instructions for use

Normasol[®] should be used by or under the supervision of a qualified health care professional.

1. Clean/swab both sides of the sachet before opening.
2. Hold sachet with longest side vertical and dotted line at the top. Open/tear sachet along the dotted line, as marked by an arrow on the sachet seal.
3. Product is single use. Discard any unused solution.

Special storage conditions and handling conditions

- Do not store the product above 25°C.
- Keep away from sunlight/keep away from heat.
- Store the sachets in the outer container (plastic pouch).
- Disposal should be handled according to local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of Normasol[®], it should be reported to Mölnlycke Health Care and to your local competent authority.

Precaution(s)

- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile, do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.
- Chemically incompatible with silver, lead and mercury.
- Do not dilute the solution before use.
- Do not use after the expiry date printed on the sachet.
- The sachet packaging is flexible. Upon opening, care in handling should be taken to avoid spillage of solution.

The Mölnlycke and Normasol trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies.



Mölnlycke Health Care AB,
Gamlestadvägen 3C, Box 13080,
SE-402 52 Göteborg, Sweden



Single sterile barrier system



Single use



Do not re-sterilize



Do not use if the product sterile barrier system or its packaging is compromised



See instructions for use



Keep away from sunlight



Upper temperature limit



Medical Device



Made in UK

www.molnlycke.com

Issued 2021-02

327197-01

Master PD-568465 rev. 03

PD-575019 rev. 03