


Product information

Can be used as
a mitt or a pad

Microfibre technology for
safe, effective and gentle
slough removal

Soft, conformable
and intuitive in use

Both sides can be used*

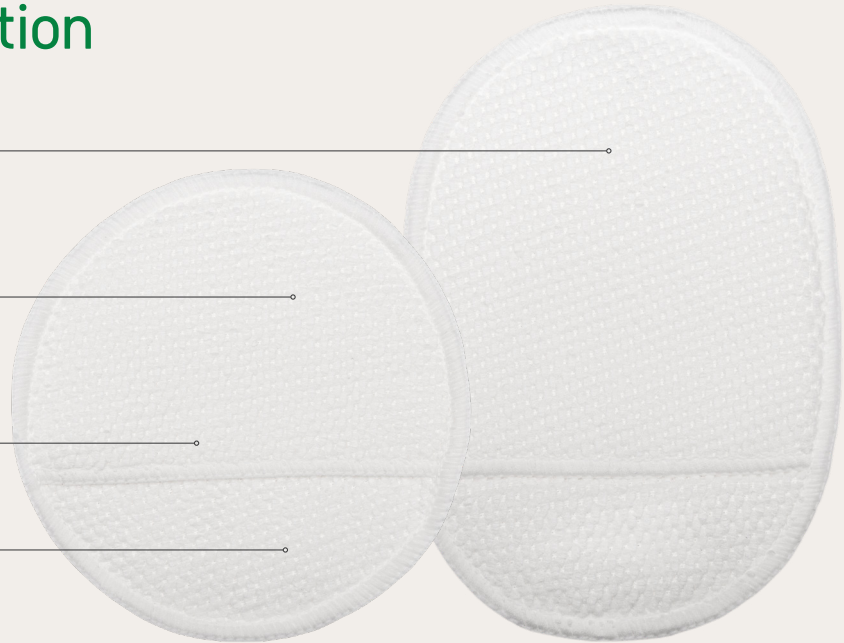

**Mepi Debripad:
designed to
be easy to use**

Simply:

- soak Mepi Debripad in saline or other suitable wound cleansing solution
- insert your hand into the mitt
- use the side of the pad to gently wipe the wound surface without applying pressure

*Mepi Debripad is for single use only. Use a new pad for each wound on the same patient. The other side (unused) of the pad can be used to cleanse peri wound skin surrounding the same wound.

Mepi Debripad assortment (Sterile packed)

	Product number	Dressing size (cm)	Pcs/box
Mepi Debripad oval	300111-00	11x18	5
Mepi Debripad	300113-00	∅ 13	5



Use the QR code
for more
information on
Mepi™ Debripad

References:

1. Harding K. et al. Simplifying venous leg ulcer management: recommendations from an expert working group (2015) Wounds International.

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, P.O. Box 6, Entreprenörstråket 21, SE-431 21 Mölndal Sweden. Phone + 46 31 722 30 00. The Mölnlycke trademark, name and logo is registered globally to one or more of the Mölnlycke Health Care Group of Companies. Mepi is a trademark of Mölnlycke Health Care AB. ©2025 Mölnlycke Health Care AB. All rights reserved. HQIM007378


Mölnlycke®

Safe and easy debridement

Designed for ease of use in any care setting



Mepi™ Debripad


Mölnlycke®

Accessible, effective debridement to promote healing

Streamlined, effective care

As a gentle, safe and effective form of debridement that is well tolerated by patients, Mepi Debripad is an important part of Mölnlycke's wound care portfolio, putting optimal wound care within reach for teams under pressure.




Debridement is proven to speed up healing for many types of wounds¹, including chronic wounds such as venous leg ulcers. It is sometimes seen as a therapy that healthcare teams under pressure may struggle to deliver. But that can change.

Mepi™ Debripad is Mölnlycke's easy-to-use microfibre pad for safe, fast and low-pain

mechanical debridement. It requires no specialist training and can be used as a standalone treatment or as a complement to sharp or chemical debridement.

With Mepi Debripad, debridement doesn't have to be a one-off surgical process but can be part of the ongoing treatment plan.

A photograph of a woman with short dark hair, wearing a light blue top, sitting on a light-colored sofa. She is looking down and to the side with a thoughtful expression. The background is a bright, out-of-focus window. A semi-transparent circular callout box is overlaid on the lower part of the image.

Professional support your team can rely on

Our wound care portfolio includes access to Mölnlycke's Clinical Learning Hub, with a comprehensive series of digital learning modules for use on demand at a time and place that suits you. It includes support for nurses who may lack confidence or experience in debridement.