Mölnlycke®

Z-flex™ Heel boot

www.molnlycke.com/zflex-heel-boot

PRODUCT FEATURES

A. Adjustable straps: Elastic straps to accommodate a variety of leg sizes (calf size: 8-18", 20-46cm).

B. Open Gate: Provides easy access to foot without removing the boot.

C. Outlast Liner: Soft and breathable liner

D. Ankle positioning straps: For adjusting foot angle

E. Aperture: Opening for the heel

F. Fluidized Positioner: For pressure redistribution over the Achilles tendon

en Mölnlycke® Z-flex™ Heel boot

Fluidized heel boot

Please read all of the instructions before use. Use as directed by appropriate clinician. Always follow your facility protocol for turning and positioning.

Intended use

The Z-flex™ Heel boot is intended for offloading the heel and to redistribute pressure from the heel and Achilles tendon to the lower leg, shin and foot.

The Z-flex™ Heel boot may be used as part of a prophylactic therapy to prevent pressure injuries.

Indications for use

Offloading heels with or without wounds in bedridden patients

Intended use environment

The product is intended to be used in hospital environment, long-term care, nursing homes and home care. The product can be used by lay persons under supervision of health care professionals.

Lifetime

The product can be used with one patient until it has deteriorated and does no longer fulfil its intended use.

Instructions for use

1. Remove the Z-flex™ Heel boot from the plastic bag.
2. Open all straps and **flatten the Fluidized positioner**. Place the heel centered over the aperture so that the Fluidized positioner cradles the Achilles tendon and the heel is offloaded.
3. Close the adjustable straps, begin at the knee and work down towards the foot. For accurate fit, slide a finger between the boot and the leg to confirm it is not too tight.
4. Attach both ankle positioning straps, leaving the foot in desired angle/position.
5. To inspect heel, detach the ankle positioning straps and open the foot gate.

All lines and tubes should exit the boot through the strap openings. Do not run any lines or tubes down the side of the leg out through the aperture at the heel.

Cleaning instructions:

Use hospital approved disinfectant or household antimicrobial cleaner / disinfectant according to the label instructions. Do not use bleach. Extended use of alcohol- based disinfectants may reduce the life of the product.

Special storage conditions/handling conditions

The Z-flex™ Heel boot should be stored below 40°C (104°F). Long exposure to high temperatures may deteriorate the product prematurely.

Warnings

- Do not stand or walk while using the boot.
- To avoid puncturing, be careful when handling sharp objects in proximity of the product.

Precautions

- Product provided non-sterile. Do not sterilize. Do not autoclave
- Do not X-ray through Fluidized heel boot.
- No maintenance required. Not user repairable.
- Replace promptly if punctured or damaged.
- If the Fluidized positioner component is punctured, place the product in a sealed plastic bag to avoid spillage of the fluidized material.

Other information

- Do not use on multiple patients (single patient use only).
- The product is CT and MRI Safe.
- Disposal should be handled according to local environmental procedures, the product can be incinerated.
- The product expiry date is printed on the packaging labels.

If any serious incident has occurred in relation to the device it should be reported to Mölnlycke Health Care.

Toll free number:
USA 1-800-882-4582
Canada 1-800-494-5134

Australian Sponsor Address:
Mölnlycke Health Care Pty. Ltd.
12 Narabang Way, Belrose, NSW 2085, Australia

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PD-540083 rev 05

Made in Mexico

The Mölnlycke trademark, name and logo are registered globally to one or more of the Mölnlycke Health Care Group of Companies

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