

# Avance® Solo Adapt

Single-use negative pressure therapy  
– without compromise

## 1. Pump

- Regulates pressure to deliver a continuous -125mmHg negative pressure for 14 days<sup>1</sup>.
- Visual and audible alarms for leakage, blockage and low battery, ensuring issues are addressed promptly.
- Portable and light weight – minimising impact on patient mobility<sup>1,3</sup>.

## 2. 50ml canister

- Canister may be replaced by clinician, carer or the patient themselves<sup>4,5</sup>.

## 3. CFM Technology™

- Controlled air inlet ensures uncompromised continuous negative pressure is delivered to the wound bed<sup>6,7,8</sup>.

## 4. Avance Solo Adapt Foam


- A single use polyurethane foam wound filler which can be cut to the shape of the wound.

## 5. Avance Solo Adapt Film with Safetac® technology

- A conformable film, Safetac soft silicone adhesive produces an effective seal for negative pressure therapy<sup>9,10</sup>.

## Consistent, regulated negative pressure

- Controlled Fluid Management (CFM) Technology enables the system to deliver consistent, regulated negative pressure to the wound site while transporting exudate from the wound to the dressing and canister<sup>1,5,8</sup>.

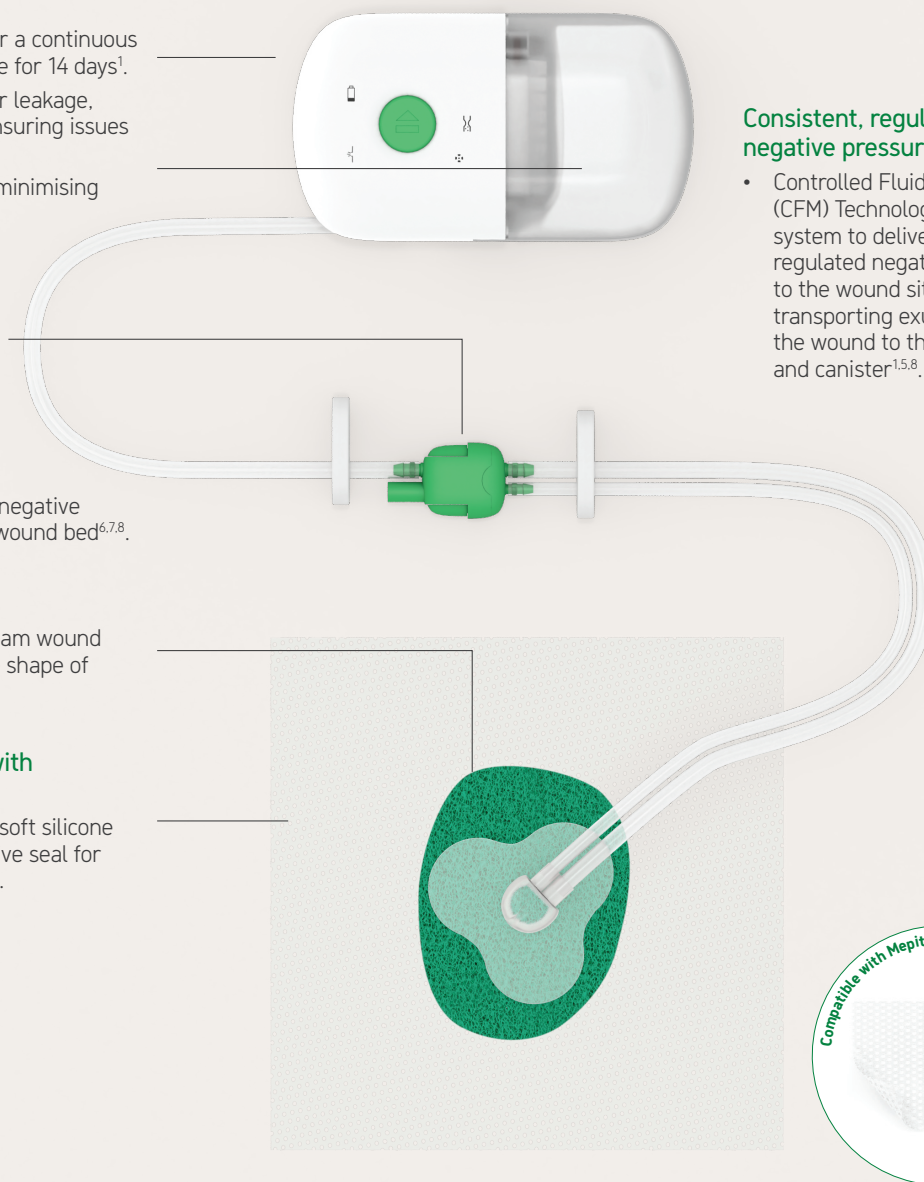

Avance Solo

Avance Solo Adapt Negative Pressure Wound Therapy (NPWT) system is indicated for use on pressure ulcers, where it aims to promote faster healing, improved patient mobility.

# What is negative pressure wound therapy?

Negative Pressure Wound Therapy (NPWT) is a method of treating wounds using pressure. The dressing is airtight and the pump applies negative pressure, clearing the wound of exudate and fluids.

## What are the benefits of Avance Solo Adapt?

CFM Technology – a combination of controlled air flow and a distal canister that enables the system to deliver continuous regulated negative pressure to the wound site, while transporting exudate from the wound to the distal canister<sup>6,7,8</sup>.

The Avance Solo Adapt pump will deliver therapy for 14 days.

The kit includes a customisable 3cm foam which can be used in conjunction with Mepitel® wound contact layer, to prevent against tissue in-growth into the foam.

The visual signals and audible alarms of the pump help to ensure that loss of therapy or system issues are promptly addressed to enable therapy to continue<sup>1,2</sup>.

## Areas of use

### dNPWT – Pressure ulcers

Avance Solo Adapt is an NPWT system indicated for use on pressure ulcers, where it is desirable/necessary to be able to customise the dressing to fit to contoured anatomy in order to achieve a seal.



## Avance Solo Adapt assortment

Code	Description	Individual Sterile Unit Contents	Retail Pack	Transport Pack
994152	Avance Solo Adapt Film Dressing Kit	Film dressing + Foam wound filler + Transfer port and tubing	5	5
994000	Avance Solo Adapt Film Dressing	5 x Safetac adhesive film dressings	5	10
9994152	Avance Solo Adapt Starter Set	Pump + Canister 50ml with tubing + 4 x AA batteries + Safetac film dressing + Foam + Belt Clip	1	2

1. Avance Solo Adapt CMM Data on file (ref 16). 2. Avance Solo Adapt CMM Data on file (ref 17). 3. Avance Solo Adapt CMM Data on file (ref 21). 4. Avance Solo Adapt CMM Data on file (ref 14). 5. Avance Solo Adapt CMM Data on file (ref 15). 6. Avance Solo Adapt CMM Data on file (ref 10). 7. Avance Solo Adapt CMM Data on file (ref 18). 8. Avance Solo Adapt CMM Data on file (ref 13). 9. Avance Solo Adapt CMM Data on file (ref 23). 10. Avance Solo Adapt CMM Data on file (ref 22).

Find out more at [molnlycke.com](https://molnlycke.com)

Mölnlycke Health Care AB, P.O. Box 6, Entreprenörstråket 21, SE-431 21 Mölndal Sweden. Phone + 46 31 722 30 00.  
The Mölnlycke, Avance, Mepitel and Safetac trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2025 Mölnlycke Health Care. All rights reserved. HQIM007596

