

How to apply Mepilex® Border Flex

How to apply
Mepilex Border Flex on a foot ulcer
with smaller size dressings.
7.5x7.5 cm
10x10 cm
12.5x12.5 cm



1. Hold the larger section of the release liner and partially peel back.



2. Position the exposed part of the dressing over the wound and apply.



3. Continue to remove the larger section of the release liner and smooth down the borders.



4. Remove the second release liner.



5. Smooth down the borders.

How to apply
Mepilex Border Flex on a leg ulcer
with larger size dressings.
15x15 cm
15x20 cm



1. Remove the middle release liner.



2. Apply to wound.



3. Remove second release liner.



4. Remove third release liner.



5. Smooth down the borders.

www.molnlycke.com.au | www.molnlycke.co.nz

Mölnlycke Health Care, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. www.molnlycke.com.au. New Zealand Orders & Enquiries 0800 005 231, www.molnlycke.co.nz. The Mölnlycke and Mepilex trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2020 Mölnlycke Health Care. All rights reserved. AUWC0220_V2


Mölnlycke®