


Mepilex® Border Flex



When to change the dressing according to saturation


The ideal wound dressing is designed to minimise the level of tissue disturbance. Too frequent dressing changes can lead to reduced temperature of the wound bed and disturbance to healing cells¹. The added benefit to an institution is a reduction in spend due to dressings being changed too often.

Saturation Levels:

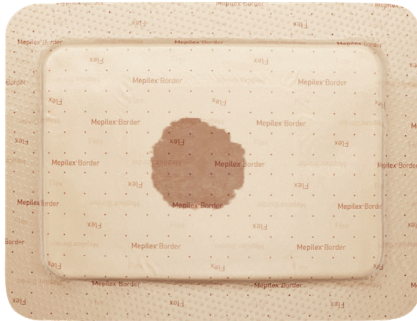


0

Strikethrough at 0 edges.
Can keep in place.

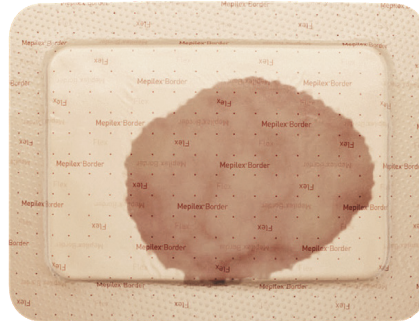


1

Strikethrough at 1 edge.
Can keep in place.

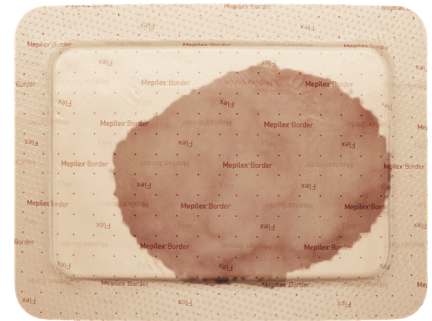

2

Strikethrough at 2 edges.
Can keep in place.

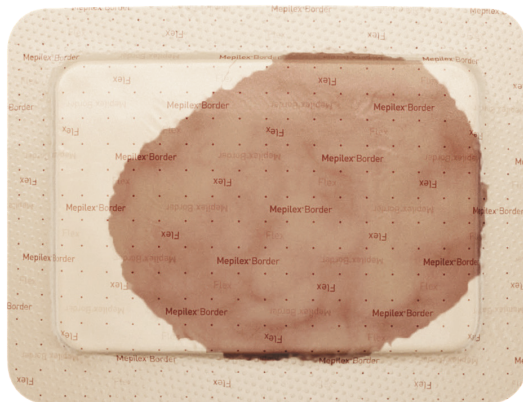

3

The dressing is saturated
when the strikethrough
has reached **3 edges**.



Mepilex Border Flex
offers up to a seven-day wear time².

Mepilex Border Flex is designed for smart exudate management. **Initial strikethrough does not necessarily mean that the dressing is saturated.** The dressing was constructed to make the strikethrough visible, so that you can determine when it is time to change it.

Mepilex Border Flex may be left in place for up to seven days depending on the condition of the wound and the surrounding skin.

All images are stylised for ease of visualisation. Exudate Progress Monitor dots do not change appearance with/without exudate.

Disclaimer: The information provided herein is not to be construed as the practice of medicine or substituted for the independent medical judgment of a patient's treating healthcare professional. Each patient's healthcare professional shall remain solely responsible for assessing the severity of patient wounds, determining the appropriate treatment and managing the treatment of the wound.

References: 1. Rippon M. et al. Taking the trauma out of wound care: the importance of undisturbed healing. Journal of Wound Care Vol 2 1 , No 8 , August 2012. 2. Nelson, D. Better Outcomes for Skin Tears with New 5 Layer Bordered Foam Dressings. Poster Presentation at WOCN Annual Meeting, Philadelphia, PA. 2018. 3. Mepilex Border Flex – Fluid handling capacity. Report no. PD-527642_01. Data on file. 4. Mepilex Border Flex—Absorption and dispersion tests on inclined plane, viscous test solution. Report no. PD-528871_02. Data on file.

www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code **1FtYrV** or your **AHPRA registration number** to access our websites. Mölnlycke Health Care, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. New Zealand Orders & Enquiries 0800 005 231, www.molnlycke.co.nz. The Mölnlycke, Mepilex and Safetac trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2022 Mölnlycke Health Care. All rights reserved. AUWC_000349

