

Mepilex® Border Post-Op

Invest **one** minute for secure fixation


Before Application

1. Cleanse and dry the wound area

Soap and detergent residues weaken adhesion.

2. Select an appropriate dressing size

The wound pad should overlap the wound by at least 1–2cm both in width and length.

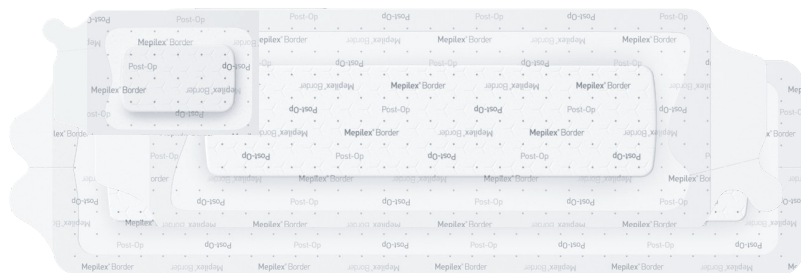

Safetac
TECHNOLOGY

Applying the dressing

1. Prepare the dressing and skin

Open the sterile packaging and remove the dressing. Check the skin is clean and dry before starting application.




2. Remove the release liner(s)

The application frame will provide extra stability whilst handling and applying the dressing to the patient. Smooth the dressing on as it is applied to ensure thorough adherence.



3. Remove the frame

Use one hand to start peeling away the frame in a downward direction. Use the other hand to smooth the border away from the wound pad as you go. Repeat on the other side.

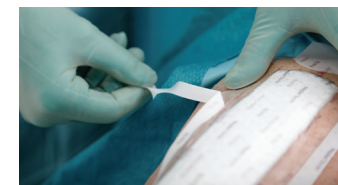


4. Secure fixation

Once in place, use your hand to smooth over the whole dressing as well as running a finger around the border for optimal adherence.



Do not change the dressing!

Change the dressing!

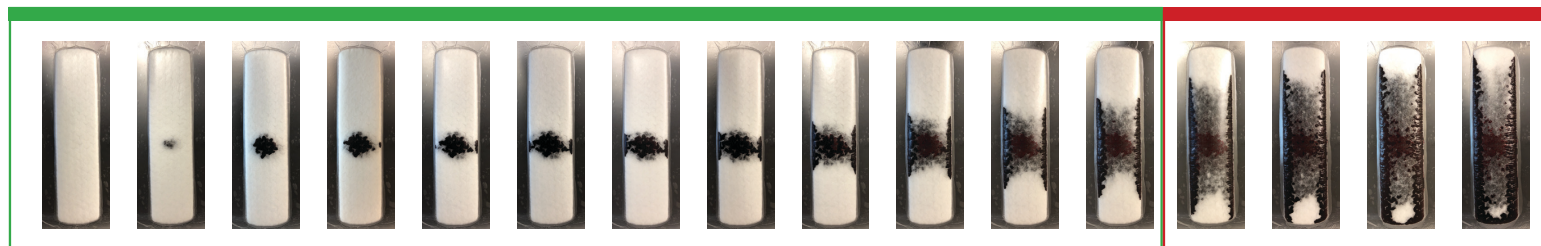

See application videos at <https://www.molnlycke.com.au/products-solutions/mepilex-border-post-op/> or <https://www.molnlycke.co.nz/products-solutions/mepilex-border-post-op/>

Find out more at www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code **1FtYrV** or your **AHPRA registration number** to access our websites.

Mölnlycke, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. New Zealand Orders & Enquiries 0800 005 231, www.molnlycke.co.nz. The Mölnlycke, Mepilex and Safetac trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. © 2024 Mölnlycke Health Care. All rights reserved. AUWC_001475


Mölnlycke®