


Mepore® Film

Breathable, transparent self-adhesive film dressing

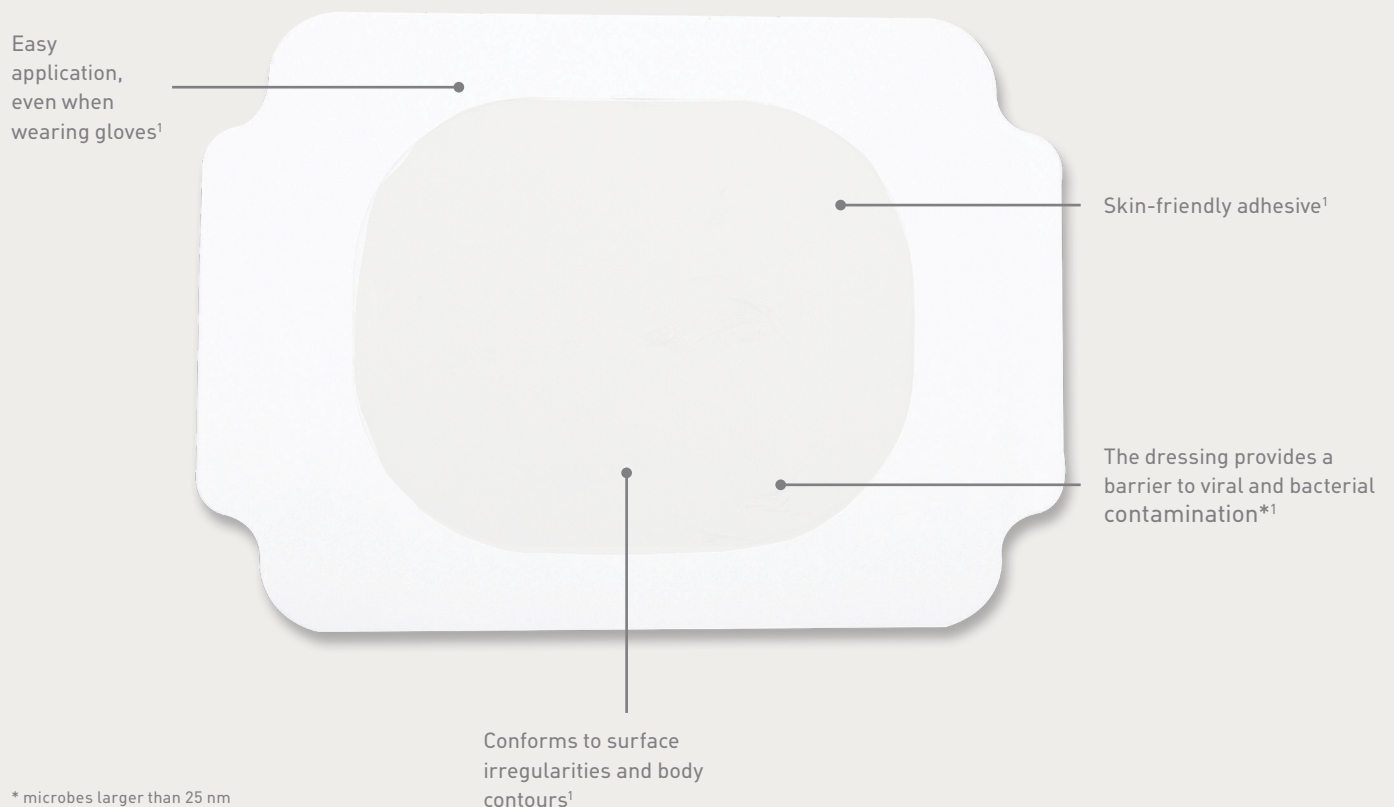

Mepore® Film is a transparent self-adhesive film dressing, ideal as a secondary fixation dressing.


Mepore Film is intended for the protection of intact skin and superficial wounds. It can also be used for fixation of medical devices, such as wound dressings, tubes and cannulas.

Owing to its high breathability, Mepore Film minimises the risk of maceration and the skin-friendly adhesive allows gentle and secure fixation.

How to use Mepore® Film



1. Remove the protection foil (printed Mepore) to expose the adhesive.

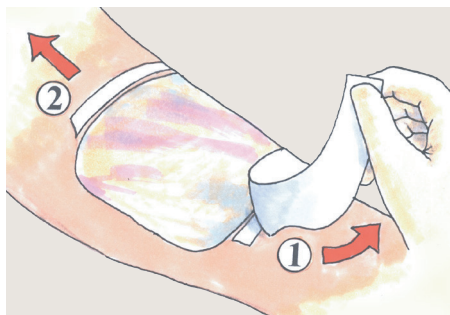

2. Position the dressing and firmly smooth it onto the skin. Remove the paper frame and the two white paper side tabs. Do not stretch the dressing when applying.

How Mepore Film works

Mepore Film provides a gentle and secure fixation. The transparency of the film allows the skin to be continuously monitored.

The film is a bacterial and viral barrier* and allows evaporation of moisture and skin transpiration.

Mepore Film is flexible and conforms to surface irregularities and body contours.

Areas of use

Mepore Film is a transparent self-adhesive film dressing, ideal as a secondary fixation dressing (ie covering and fixating Mesorb).

It is intended for the protection of intact skin and superficial wounds. Mepore Film can also be used for fixation of medical devices, such as wound dressings, tubes and cannulas.

* microbes larger than 25 nm

Benefits of Mepore Film

- Secure fixation
- Gentle to the skin
- The film is transparent
- Thin and flexible
- Conforms to body contours
- High breathability
- Easy to use
- Aseptic application possible



Mepore Film is packaged sterile in single packs

Product Code	Size (cm)	Pcs/Box
270600	6 x 7	100
271500	10 x 12	70
272500	10 x 25	20
273000	15 x 20	10

Reference: 1. Mölnlycke Health Care. Data on file. 2021.

www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code **1FtYrV** or your **AHPRA registration number** to access our websites.
Mölnlycke, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. New Zealand Orders & Enquiries
0800 005 231, www.molnlycke.co.nz. The Mölnlycke and Mepore trademarks, names and logos are registered globally
to one or more of the Mölnlycke Health Care Group of Companies. ©2022 Mölnlycke Health Care. All rights reserved. AUWC_000314

