


Mepore® Film & Pad

Breathable, transparent self-adhesive absorbent film island dressing

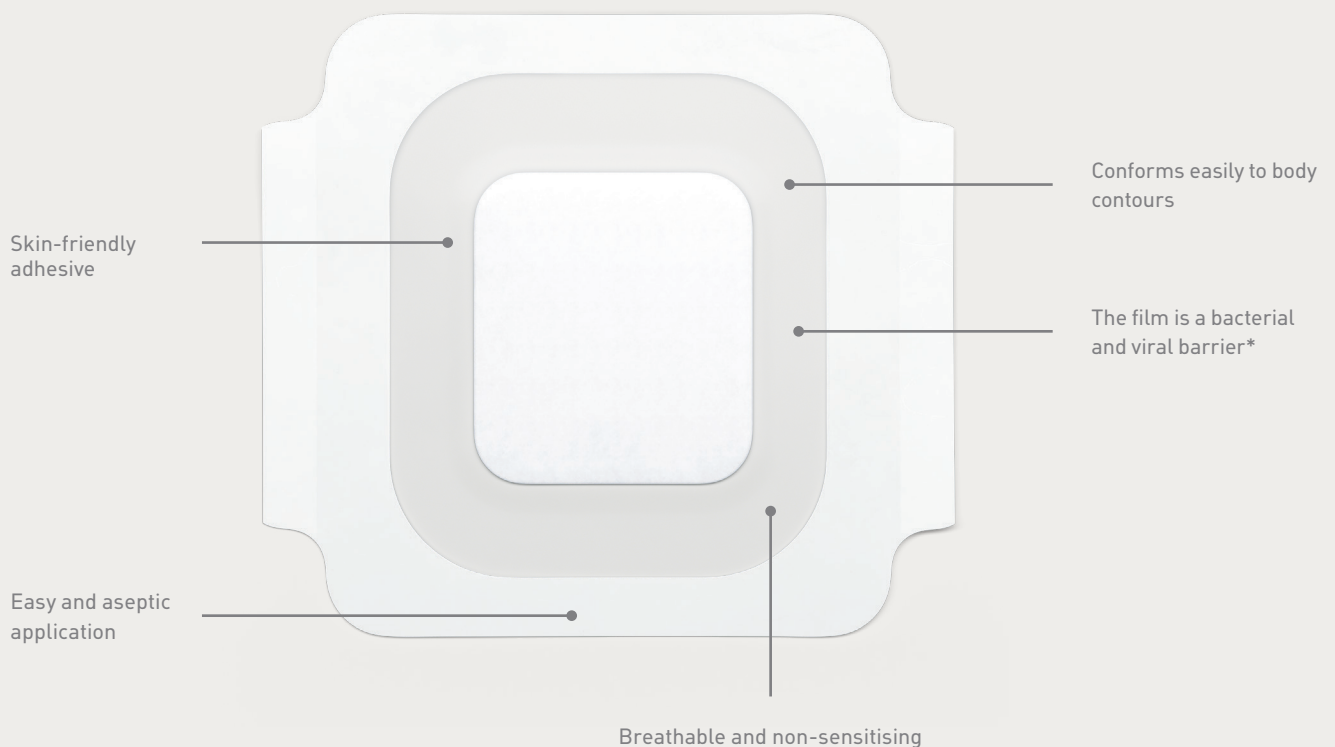

* microbes larger than 25 nm


Mepore® Film & Pad is intended for use on low to moderately exuding wounds, such as minor surgical wounds, cuts and abrasions.

It stays in place on the patient, allows easy and gentle removal and is showerproof.

How to use Mepore® Film & Pad



1. Remove the release film. Position the dressing. Do not stretch the dressing when applying.

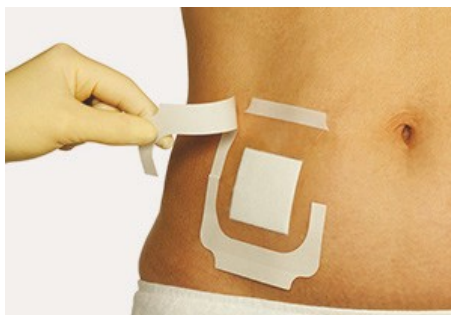

2. When applying the dressing, remove the paper frames and the two white tabs and smooth out the borders to remove any creases and wrinkles.



3. To ensure proper adhesion, put your hand over the dressing for a few seconds.

How Mepore Film & Pad works

Mepore® Film and Pad is a breathable, transparent self-adhesive absorbent film island dressing, ideal for absorption of blood and exudate. It protects the wound and can be used for several days.

The skin-friendly adhesive allows a gentle fixation. It is easy to apply and change aseptically, even when wearing gloves.

There is a wide assortment of sizes.

Benefits of Mepore Film & Pad

- Stays in place
- The film is a bacterial and viral barrier*
- Flexible and transparent film
- Conforms easily to body contours
- Aseptic application possible
- Allows showering
- Low-adherent wound contact layer

* microbes larger than 25 nm



Areas of use

Mepore Film & Pad is intended for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

It absorbs blood and exudate and protects the wound. The showerproof film is impermeable to fluids and breathable.

Frequency of change

The frequency of dressing changes will be governed by the condition of the underlying wound. Mepore Film & Pad can be used for several days.

Mepore Film & Pad is packaged sterile in single packs.

Product code	Size (cm)	Wound pad size (cm)	Pcs/Box
275100	4 x 5	1.5 x 2.5	85
275200	5 x 7	2.5 x 4 (oval)	85
275300	5 x 7	2.5 x 4	85
275400	9 x 10	4.5 x 5	30
275500	9 x 15	4.5 x 10	30
275600	9 x 20	4.5 x 15	30
275700	9 x 25	4.5 x 20	30
275900	9 x 35	4.5 x 30	25

www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code 1FtYrV or your AHPRA registration number to access our websites.

Mölnlycke, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. New Zealand Orders & Enquiries 0800 005 231, www.molnlycke.co.nz. The Mölnlycke and Mepore trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2022 Mölnlycke Health Care. All rights reserved. AUWC_000313

