

Mepore® Pro

Shower-proof, self-adherent absorbent dressing

Easy-to-remove
protection paper
for ease of use.

Absorbent wound
pad with a low-
adherent wound
contact layer for
better patient
comfort.

Skin-friendly,
water-based and
solvent-free
adhesive for gentle
and secure fixation.

Soft, elastic nonwoven for
better patient comfort.

The backing film is a
bacterial and viral barrier*
for protection of the wound
from outer contamination.

Breathable and shower-
proof film for better
patient comfort.

* Microbes larger than 25 nm

Mepore Pro is a self-adherent, absorbent and breathable dressing with an outer film layer which protects the wound from water and contamination. Its skin-friendly, water-based and solvent-free adhesive provides gentle and secure fixation.

How to use Mepore® Pro



1. Grip both overlapping release papers, exposing adhesive surface to allow initial fixation of the dressing.



2. Position the dressing on the skin without stretching and gently remove the rest of the release papers.



3. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying.

How Mepore Pro works

The backing film allows showering and protects the wound from water and contamination. It also protects clothing and bed linens from blood or exudate strikethrough. Due to a highly absorbent pad and a high vapour permeability of the backing film, the dressing offers a good total fluid handling.

The polyacrylate adhesive is designed to be gentle to the skin while providing secure fixation. The easy-to-remove protection paper permits sterile application, and simplifies dressing application. The stability of the fabric / film laminate also supports ease of application.

Benefits of Mepore Pro

- Allows showering.
- Low-adherent wound contact layer minimises risk of adherence to the wound.
- Protects wound from water and outside contamination
- The backing film is a bacterial and viral barrier*.
- Optimised absorption capacity.
- Easy-to-remove protection paper allows for convenient and aseptic application.
- Breathable film.
- Low friction to clothing and bed linens.
- Skin-friendly water-based adhesive.
- Rounded corners for secure fixation (all sizes up to 9x15cm).

* microbes larger than 25 nm

Areas of use

Mepore Pro can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, cuts, and abrasions.

Frequency of change

The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal, grasp the edge and slowly peel the dressing back from the skin in the direction of hair growth.

Precautions

Care should be taken to ensure that Mepore Pro is not applied under tension, to prevent shearing forces from causing damage to the skin. This is particularly important when the dressing is applied over joints.



Ordering information (sterile packaged)

Product Code	Size (cm)	Wound pad size (cm)	Pcs/Box
670820	6 x 7	3 x 4	60
670920	9 x 10	4.5 x 6	40
671020	9 x 15	4.5 x 10	40
671120	9 x 20	4.5 x 15	30
671220	9 x 25	4.5 x 20	30
671320	9 x 30	4.5 x 25	30

www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code **1FtYrV** or your **APHRA registration number** to access our websites.

The Mölnlycke, Mepore names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2021 Mölnlycke Health Care. All rights reserved. Mölnlycke Health Care, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. www.molnlycke.com.au. New Zealand Orders & Enquiries 0800 005 231, www.molnlycke.co.nz. AUWC_IM000425_V2

