


Skin tear algorithm of care

Mölnlycke® has developed a simple classification guide to skin tears in accordance with the proposed standardisation of identifying and documenting skin tears produced by ISTAP.

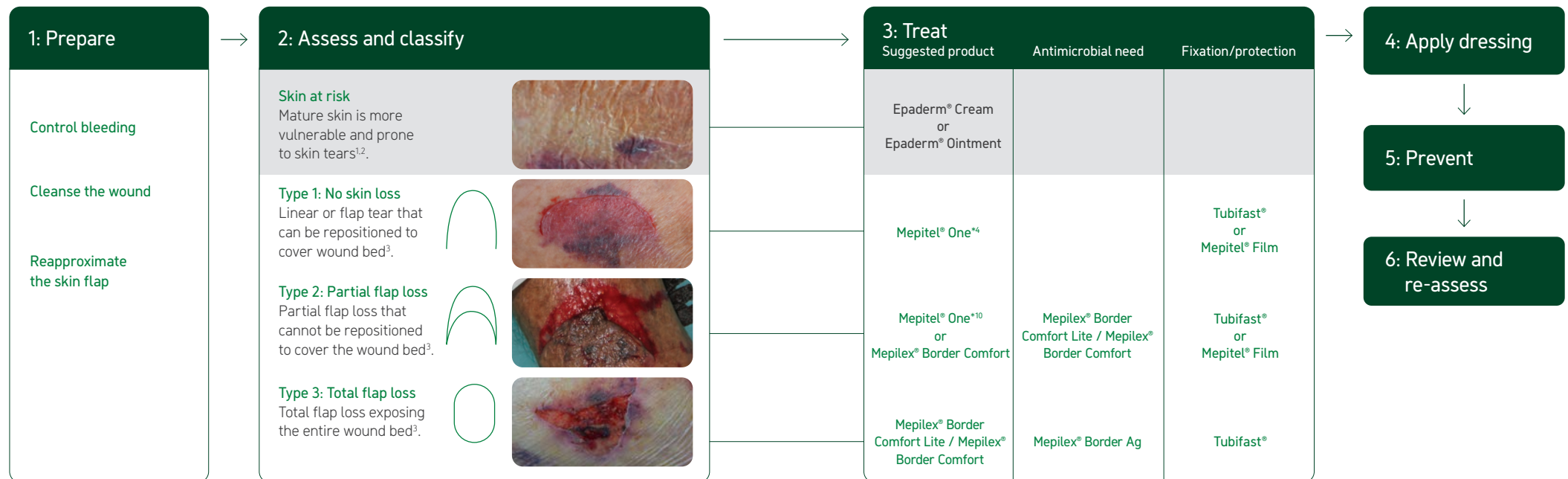

Scan the QR code to read more about skin tears management or visit <https://www.molnlycke.co.uk/wound-types/skin-tears/>



References:

1. White MW. et al. Skin tears in frail elders: A practical approach to prevention. Geriatric Nursing, 1994.
2. Resnick B. Wound care for the elderly. Geriatric Nursing, 1993.
3. LeBlanc K. et al. International skin tear advisory panel: A tool kit to aid in the prevention, assessment, and treatment of skin tears using a simplified classification system. Advances in Skin and Wound Care, 2013.
4. Woo K. & LeBlanc K. A randomised controlled pragmatic study to evaluate the use of silicone dressings for the treatment of skin tears. Poster presentation at Wound Con Summer (virtual) conference, 2020.

Find out more at www.molnlycke.co.uk

Mölnlycke Health Care Limited: Unity House, Medlock Street, Oldham, Lancashire, OL1 3HS, United Kingdom. Phone: +44 (0) 161 621 3900. The Mölnlycke, Mepitel, Mepilex, Tubifast and Epaderm trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2025 Mölnlycke Health Care AB. All rights reserved. UKWC1334



Skin tear algorithm of care

Mölnlycke® has developed a simple classification guide to skin tears in accordance with the proposed standardisation of identifying and documenting skin tears produced by ISTAP.



Scan the QR code to read more about skin tears management or visit <https://www.molnlycke.co.uk/wound-types/skin-tears/>

Mepilex® Border Comfort

MPC	Size	UOI	PIP Code	NHSSC
596211	7.5 x 7.5	10	407-4415	ELA1103
596311	10 x 10	10	407-4423	ELA1104
596011	12.5 x 12.5	10	407-4431	ELA1102
596411	15 x 15	10	407-4449	ELA1105
596800	10 x 20	10	413-0472	ELA85021
596900	10 x 30	10	413-0480	ELA85020
596611	15 x 20	10	407-4456	ELA1106

Epaderm®

MPC	Size	UOI	PIP Code	NHSSC
99400816	Ointment 125g x 6	1	015-9111	ELZ267
99400817	Ointment 500g x 6	1	027-0652	ELZ268
99400818	Ointment 1kg x 6	1	339-4822	ELZ269
99400820	Cream 500g	1	343-2549	ELZ368
99400821	Cream 50g	1	343-2531	ELZ265

Mepitel® Film

MPC	Size	UOI	PIP Code	NHSSC
296800	6.5 x 7	10	373-4159	EIJ084
296500	10.5 x 12	10	373-4167	EIJ085
296700	10.5 x 25	10	373-4175	EIJ086
296900	15.5 x 20	10	373-4183	EIJ087

Mepilex® Border Comfort Lite

MPC	Size	UOI	PIP Code	NHSSC
582011	4 x 5	10	424-0404	ELA85015
582111	5 x 12.5	10	424-0412	ELA85016
582211	7.5 x 7.5	10	424-0438	ELA85017
582311	10 x 10	10	424-0420	ELA85018
582511	15 x 15	10	424-0396	ELA85019

Mepilex® One

MPC	Size	UOI	PIP Code	NHSSC
289170	6 x 7	5	350-6888	EKH037
289270	9 x 10	5	350-6896	EKH038
289470	13 x 15	5	350-6904	EKH039
289670	24 x 27.5	5	350-6912	EKH040
289750	27.5 x 50	2	403-1894	EKH0769

Mepilex® Border Ag

MPC	Size	UOI	PIP Code	NHSSC
395260	7 x 7.5	5	362-3824	ELA516
395360	10 x 12.5	5	362-3873	ELA517
395460	15 x 17.5	5	362-3840	ELA518
395660	17 x 20	5	362-3832	ELA519
395800	10 x 20	5	362-3881	ELA520
395700	20 x 25	5	362-3865	ELA777
395900	10 x 30	5	362-3857	ELA522

Find out more at www.molnlycke.co.uk

Mölnlycke Health Care Limited: Unity House, Medlock Street, Oldham, Lancashire, OL1 3HS, United Kingdom. Phone: +44 (0) 161 621 3900. The Mölnlycke, Mepitel, Mepilex, Tubifast, Mesoft, Epaderm and Mesorb trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2025 Mölnlycke Health Care AB. All rights reserved. UKWC1334

