

Granudacyn[®]

Wound Wash Solution

Wound cleanser

Wound cleanser with Hypochlorous acid

- For cleansing, irrigating, and moistening acute, chronic and contaminated wounds¹
- Contains Hypochlorous acid/Sodium Hypochlorite that acts as a preservative, preventing microbial growth in the solution²
- Has been shown to be safe and effective for wound cleaning³

Biocompatibility

- Non-irritating to wounds and surrounding skin due to its biocompatibility and non-cytotoxicity⁴


Ease-of-use

- Can be used up to 60 days after the bottle opening, and has 28 months shelf life⁵
- Available in various packaging including spray cap bottle, screw cap (pour) bottle, and spikeable cap (instillation) bottle
- Can be used in combination with other Mölnlycke wound dressings⁶




How to use Granudacyn[®] Wound Wash Solution

- Granudacyn can be used **multiple times**, up to **60 days after opening the bottle**.
- Granudacyn Wound Wash Solution **does not require neutralization or rinsing**.

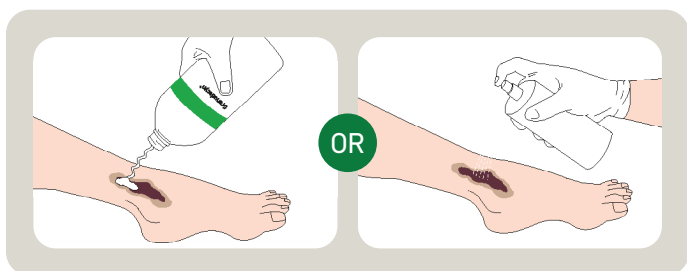

To clean and prepare the
wound bed

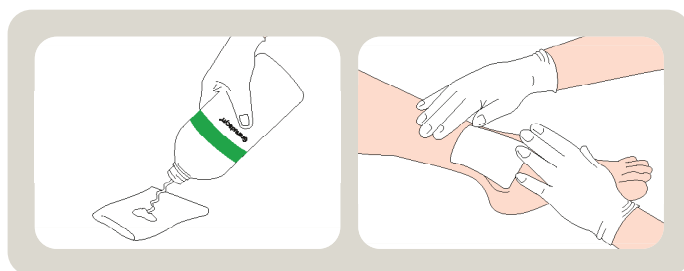

To moisten the wound dressing or
the wound itself

What it is & how it works

Granudacyn contains Hypochlorous acid/Sodium Hypochlorite that act as a preservative, preventing microbial growth in the solution, thus ensuring the safety of the product (up to 60 days after opening).² The cleansing effect of Granudacyn is based on mechanical removal of foreign material including microorganisms and debris.

Granudacyn Wound Wash solution creates a moist environment and loosens contaminated exudate, slough and other foreign materials within the wound bed.⁸

Areas of use

Under the supervision of healthcare professionals, Granudacyn Wound Wash Solution can be used for cleansing, moistening, debridement, and removal of foreign material (including microorganisms and debris) from exudating and/or dirty wounds including:

- Stage I-IV pressure injuries
- Leg and foot ulcers
- Post-surgical wounds
- First and second-degree burns
- Grafted and donor sites

Granudacyn® is part of Mölnlycke Wound Care solutions



Granudacyn® Wound Wash Solution Assortment

Product Code	Product Description	Units per Box	Units per Case	HCPCS
360170	250ml (8.5 fl oz) Spray cap	1	15	A6260
360171	500ml (16.9 fl oz) Screw cap (Pour bottle)	1	12	A6260
360172	1000ml (33.8 fl oz) Screw cap (Pour bottle)	1	6	A6260
360173	500ml (16.9 fl oz) Spikeable cap (Instillation bottle)	1	12	A6260
360174	1000ml (33.8 fl oz) Spikeable cap (Instillation bottle)	1	6	A6260

References

1. Intended use as cleared in the 510(K) K243415 2. (internal documentation) DI-7-010 Shelf Life FL100.MDD.FDA Cleared in the 510(K) K243415 3. Intended use as cleared in the 510(K) K243415 4. DI-7-372-0 Biocompatibility Report Wound Wash Solution FDA 5. (internal documentation) DI-7-010 Shelf Life FL100.MDD.FDA Cleared in the 510(K) K243415 6. Mölnlycke wound dressing such as Mepilex XT, Mepilex Lite, Mepilex Transfer Mepilex Border Lite, Mepilex Border Flex, Mepilex Flex Lite, Mepilex Ag, Mepilex Border Ag and Mepilex Transfer Ag, Internal Mölnlycke testing 7. 510(K) clearance K243415, Multiple times Dependent on container format and wound size, not applicable for Negative Pressure Wound Therapy 8. Intended use 510(K) clearance K243415

1-800-843-8497 | www.molnlycke.us | 5445 Triangle Pkwy, Ste 400, Peachtree Corners, GA 30092

Distributed by Mölnlycke Health Care US, LLC, Peachtree Corners, Georgia 30092. © 2026 Mölnlycke Health Care AB. All rights reserved. 1-800-882-4582. Mölnlycke Health Care wound care products can serve as integral components of wound management programs.

