

Mepilex® Up

Non-Bordered foam that minimizes the risk of leakage

Low profile polyurethane foam pad

- Manages high fluid volumes for a long period of time¹
- Highly conformable

Dimpled surface pattern


- Spreads exudate in all directions, even under compression and against forces of gravity^{1,2}

Polyurethane backing film

- Breathable and water resistant
- Secures fluid and evaporates excess moisture

Safetac® layer

- Reduces pain during dressing changes³
- Adheres to dry skin but not to the wound bed
- Seals wound margins and reduces the risk of maceration⁴

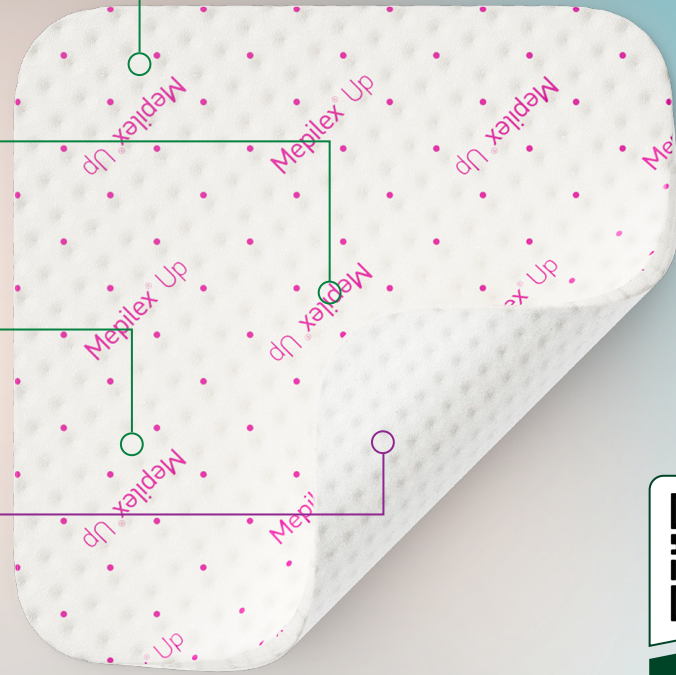


Scan here to learn more

How Mepilex Up works

The soft silicone **Safetac® layer** seals the wound margins, channeling exudate vertically into the foam and helping prevent maceration.

Stage 1

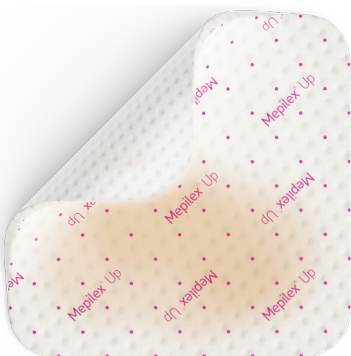


The patented foam structure creates a proprietary dimpled surface pattern.⁵

Stage 2



The dimpled pattern enables the dressing to spread fluids evenly in all directions, even against gravity.


Stage 3

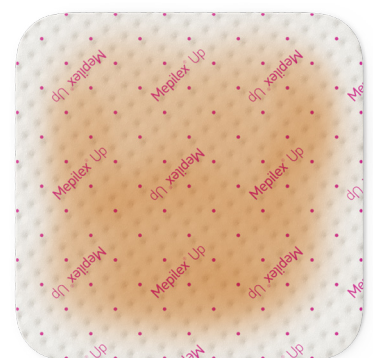

The innovative and unique spreading properties minimize the risk of leakage and may contribute to long wear time and fewer dressing changes⁶

Benefits of Mepilex® Up

- Absorbs both low and high viscous exudate
- Higher fluid handling capacity than other foam dressings⁷
- Minimizes risk of leakage, even under compression¹
- Long wear time and fewer dressing changes¹
- Promotes patient comfort during wear⁸
- Trimmable and conformable to suit various wound shapes and difficult-to-dress areas⁹
- Minimizes pain and trauma at dressing changes
- Can remain in place during X-ray, CT and MRI¹⁰

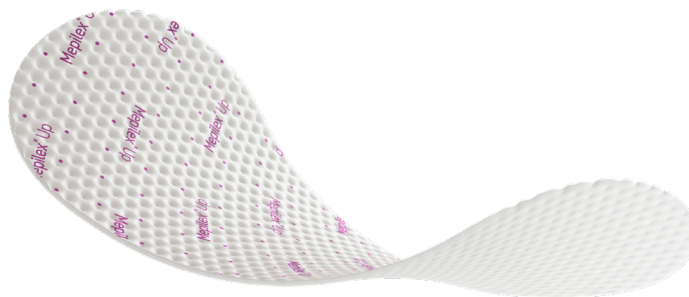

Wear Time

Mepilex Up may be left in place for several days depending on the condition of the wound and surrounding skin or as indicated by accepted clinical practice.

Indications for Use

Mepilex Up is intended for the management of a wide range of exuding wounds in all healing stages such as chronic wounds, e.g. leg ulcers, foot ulcers and pressure ulcers, and traumatic wounds, e.g. skin tears and secondary healing wounds.

Mepilex® Up may be used with these Mölnlycke Wound Management products



Exufiber® Ag+
Gelling Fiber



Mextra®
Superabsorbent



Tubigrip®
Tubular Support Bandage



Mepilex® Up ordering information

Product Code	Size	Pcs/box	Pcs/case	HCPCS
212199	4" x 4" (10 x 10 cm)	10	40	A6209
212299	4" x 8" (10 x 20 cm)	10	30	A6210
212399	6" x 6" (15 x 15 cm)	10	30	A6210
212499	8" x 8" (20 x 20 cm)	10	30	A6211

References: 1. Mölnlycke Health Care. Test report PD-701547. Data on file. 2. Mölnlycke Health Care. Test report PD-701546. Data on file. 3. Gee Kee E.L. et al. Randomized controlled trial of three burns dressings for partial thickness burns in children. Burns, 2014. 4. Wiberg A.B. et al. Preventing maceration with a soft silicone dressing: in-vitro evaluations. Poster presented at the 3rd Congress of the WUWHS, Toronto, Canada, 2008. 5. Mölnlycke Health Care Data on file 2022. 6. Mölnlycke Health Care Data on file 2022. 7. Mölnlycke Health Care. Test Report PD-741994. Data on file 2024. 8. Weaver, G., Crawford, K. The use of Mepilex on Dehiscent Amputation Wounds. Wounds UK 2007; 3(1). 9. Mölnlycke Health Care Data on file 2024. 10. Mölnlycke Health Care Data on file. 2024.

Find out more at www.molnlycke.us

The Mölnlycke Health Care, Mepilex, Exufiber, Mextra, Tubigrip and Safetac trademarks, names and logo types are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Distributed by Mölnlycke Health Care US, LLC, Peachtree Corners, Georgia 30092. © 2025 Mölnlycke Health Care AB. All rights reserved. 1-800-843-8497.
USMM_270758 04.25

MHC-2025-000060

